

Baby Umbi

Directions for Use



Laerdal™
helping save lives

Table of Contents

| | |
|----------------------|---|
| Laerdal Recommends | 4 |
| Items Included | 4 |
| Skills Taught | 4 |
| Instructions for Use | 4 |
| Care and Maintenance | 5 |
| Replacement Parts | 5 |

Baby Umbi (250-00501) is a full body, lifelike reproduction of a newborn infant. It is specifically designed for training professionals in the practice of umbilical cord catheterization.

The manufacturing excellence of this trainer should provide many training sessions when reasonable care and maintenance is practiced.

Laerdal Recommends:

Umbilical catheter – Size 3

Items Included:

- (1) Baby Umbi
- (3) Umbilical Cords
- (1) Bottle of Simulated Blood Concentrate
- (1) Laerdal Global Warranty Booklet
- (1) Directions for Use

Skill Taught:

Umbilical cord catheterization

Instructions:

1. Fill abdominal reservoir by pouring blood formula through umbilical opening.
2. Insert cord into umbilical opening.
3. Several preliminary catheter insertions may be required before cord vessels become sufficiently lubricated with blood formula.
4. Drain abdominal reservoir and cord before storage.

**Laerdal
Recommends**

Insertion will require lubrication with full strength liquid soap.

When the end of the cord being catheterized is no longer useable, cut off ¼ inch and continue to use.

Care and Maintenance:

1. Clean with mild soap and water; do not submerge in cleaning fluids or water.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the trainer on newsprint or inked lines of any kind.
3. To ensure longevity, each trainer should be cleaned after each training session and a general inspection should be conducted regularly.
4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed.
5. Articulating parts will benefit from a light application of talcum powder prior to training sessions.
6. Store properly between teaching sessions.



Cautions and Warnings

This product contains **Natural Rubber** latex which may cause allergic reactions when in contact with humans.

Replacement Parts:

Product Number:

250-00501
240-00150
240-00250

Description:

Baby Umbi
Cord, Umbilical Set (3)
Baby Umbi Simulated
Blood Concentrate

©Laerdal Medical, Gatesville TX. All rights reserved
1001656 rev B
October 31, 2006