


Jose Monte
CONTROLLED
7/15/06

REV	DESCRIPTION OF CHANGE	CHANGE REFERENCE	ORIGINATOR	VERIFIED	APPROVED
			DATE AND SIGN	DATE AND SIGN	DATE AND SIGN
B	Remove Global Address List	LMAS	<i>7/13/06</i> JMa 7/15/06	MY 7/15/06	SLa 7/15/06
A	Release	N/A	DATE AND SIGN	DATE AND SIGN	DATE AND SIGN
			MY 12/29/2005	JM 12/29/2005	SL 12/29/2005
THE INFORMATION CONTAINED HEREIN IS PROPRIETARY TO LAERDAL AND SHALL NOT BE USED FOR ANY PURPOSE DETRIMENTAL TO OR BE REPRODUCED WITHOUT PERMISSION OF LAERDAL		DMR <input checked="" type="checkbox"/>	 Laerdal <i>helping save lives</i>		
		DHF <input type="checkbox"/>			
REFERENCE	N/A	REPLACES	N/A	REV	N/A
DFU Fundus Skills & Assessment Task Trainer					
PRODUCT GROUP	TOOL	PART NO	REV	DOCUMENT NO	PAGE
TT-GEN	N/A	1004885	A	PRO-ML03-0107	1 of 1
				ATTACH	REV
				1	B

Fundus Skills and Assessment Task Trainer

Directions for Use



Laerdal[™]
helping save lives

Table of Contents

Items Included	4
Features	4
Directions for Use	5
Care and Maintenance	5
Replacement Parts	5

The Fundus Skills and Assessment Task Trainer (375-22501) is designed for training in postpartum care.

The manufacturing excellence of this trainer should provide many sessions of training when reasonable care and maintenance is practiced.

Items Included:

- (1) Fundus Skill Assessment Trainer
- (1) Internal Components (Platform Fundus Stomach, Fundus Small Ball & Fundus Large Ball)
- (1) Genitalia
- (1) Simulated Blood Concentrate
- (5) Peri Pads
- (1) Talcum Powder
- (1) Laerdal Global Warranty Booklet
- (1) Directions for Use

Features:

- Normal anatomy of the postpartum female abdomen
- Landmark of the symphysis pubis
- Umbilicus
- Fundus offset secondary to full bladder
- Anal hemorrhoids visible
- 2nd degree midline episiotomy
- Simulates either firm or boggy fundus
- Fundus consistencies:
 - Approximately a 3” diameter ball to simulate a firm, well contracted uterus
 - Approximately a 4” ball to simulate a “boggy” uterus that is not contracted
- Genitalia elongated with swelling of clitoris area
- Vaginal orifice is visible and open
- Labium minus flapped to expose vaginal canal

Directions for Use:

1. Apply baby powder to inner skin.
2. Insert ball to simulate “firm” or “boggy” uterus.

Care and Maintenance:

1. Clean with mild soap and water; do not submerge modules or parts in cleaning fluids or water.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, manikin and module should be cleaned after each training session and a general inspection should be conducted regularly. A generous dusting of powder will assist in protecting modules and promoting longevity.
4. Store properly between teaching sessions.

Replacement Parts:

Part Number:

375-22650

375-22750

375-22850

375-22950

375-23050

375-21250

375-21150

325-00150

325-00250

375-23150

375-23250

277-00150

300-00750

375-23350

Part:

Pelvis Skin

Urinary Bladder

Reservoir Assembly

Female Fundus

Genitalia Assembly

Urinary Valve

Connector

Pubis Bone Pelvis Pin

Right Stump

w/Injection Pad

Left Stump w/Injection
Pad

Ventral Gluteal Pad

Dorsal Gluteal Pad

Small Foam Fundus Ball

Large Foam Fundus Ball

Baby Powder

Blood Concentrate

Peri Pads (5)

Please contact Customer Service Representatives for more information on Replacement Parts in other skin tones.

(C) Copyright Laerdal Medical, Gatesville TX. All rights reserved
1004885 rev C
August 30, 2006