

Pediatric Intubation Trainer

Directions for Use



Laerdal[™]
helping save lives

Table of Contents

Items Included	4
Skills Taught	4
Laerdal Recommends	5
Instructions for Use	5
Care and Maintenance	6
Replacement Parts	7

The Pediatric Intubation Trainer (255-00001) realistically simulates the head and upper body of a six-year-old hospital patient. It is specifically designed for training professionals in the field of pediatric airway management techniques.

The manufacturing excellence and ease of replacing individual parts of this trainer should provide years of training when reasonable care and maintenance is practiced.

Items included:

- (1) Pediatric Intubation Trainer
- (1) Manikin Lubricant
- (1) Shorts
- (1) Carry Case
- (1) Laerdal Global Warranty Booklet
- (1) Directions for Use

Skills taught:

- Oral intubation
- Nasal intubation
- Digital intubation
- Bag/Valve Mask
- Oropharyngeal airway insertion
- Naso pharyngeal airway insertion

Laerdal Recommends

Endotracheal tube size 4 or 5, cuffed or uncuffed

Manikin lubricant or liquid soap

Laryngoscope blade, size 2 straight or curved blade

Laryngoscope handle

Instructions for Use:

For intubation of the pediatric intubation head, the following equipment is recommended.

- a. Endotracheal tube size 4 or 5, cuffed or uncuffed
- b. Manikin lubricant or liquid soap
- c. Laryngoscope blade, size 2 straight or curved blade
- d. Laryngoscope handle

Laerdal Recommends

Prior to airway placement, all tubes and passages must be lubricated with a spray of Manikin Lubricant or mild liquid soap.

Care and Maintenance:

1. Clean with mild soap and water
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, each manikin should be cleaned after each training session and a general inspection should be conducted regularly.
4. All parts should be drained and air-dried thoroughly before storage and disinfected when needed.
5. Store properly between teaching sessions.

Replacement Parts:

Product Number:

231-00250

255-00250

255-00150

231-00350

255-00450

300-00650

255-00350

276-00150

Description:

Pediatric Intubation

Head

Chest Skin

Chest Foam Insert

Pediatric Simulated

Lungs

Shorts

Manikin Lubricant

Bulb Assembly

Carry Case

©Laerdal Medical, Gatesville TX. All rights reserved
1001635 rev B
December 19, 2006