







# Resusci Junior QCPR

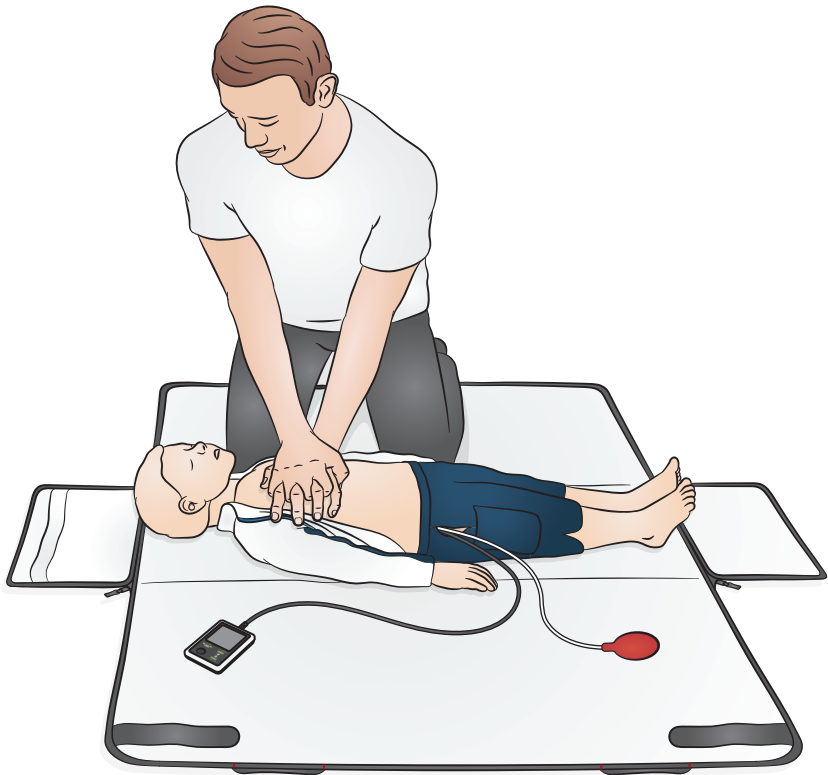


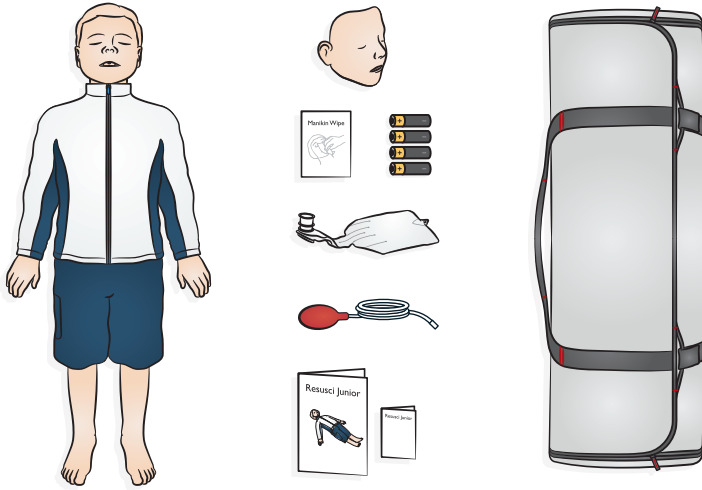

EN

Resusci Junior QCPR is a member of the Resusci QCPR range of training manikins, designed for High Quality Child CPR training. To enable real time feedback and debriefing, connect to SkillGuide, SimPad or any device with QCPR Learner/Instructor app installed.

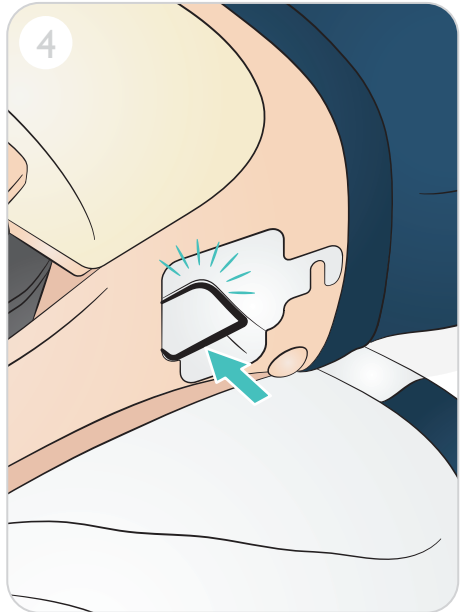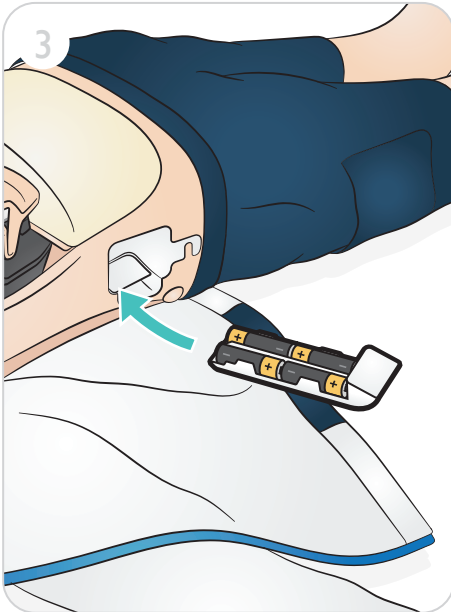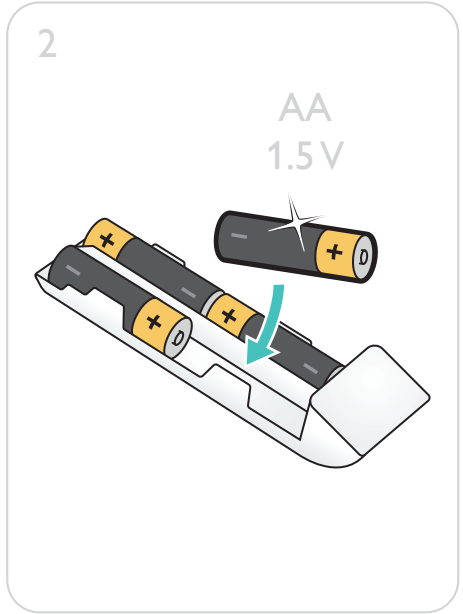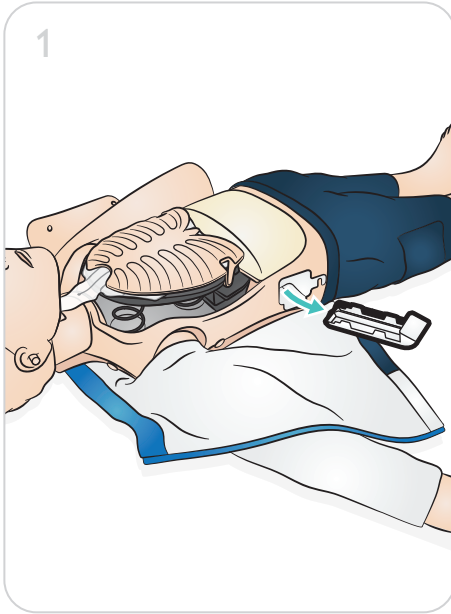

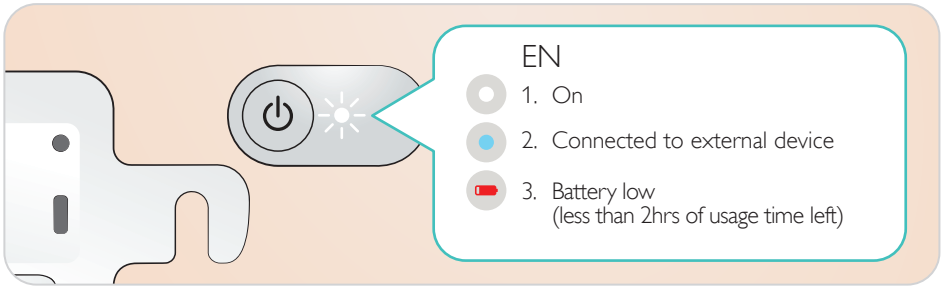A smartwatch face is shown on the left. A notification bubble is overlaid on the watch face, containing a list of items. The bubble has a pointer pointing to a sun icon on the watch face.

EN

1. On
2. Connected to external device
3. Battery low  
(less than 2hrs of usage time left)



EN

SkillGuide  
connection

2

1

On/Off button, chest  
compressions will also  
turn on manikin

3

Pulse bulb  
connection

6


Cable  
holder

4

Battery  
compartment

5


External power  
(USB-C port)

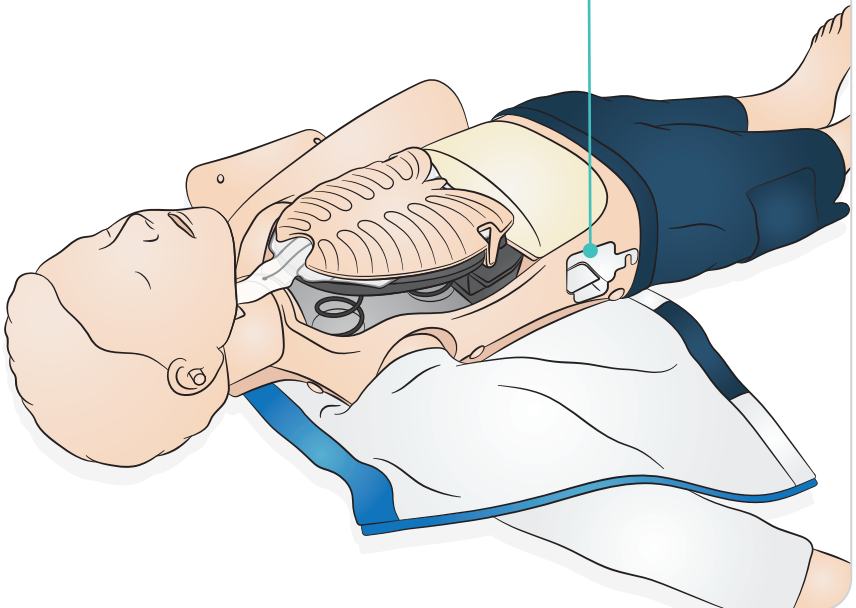




Apple  
QCPR Learner  
QCPR Instructor

Android  
QCPR Learner


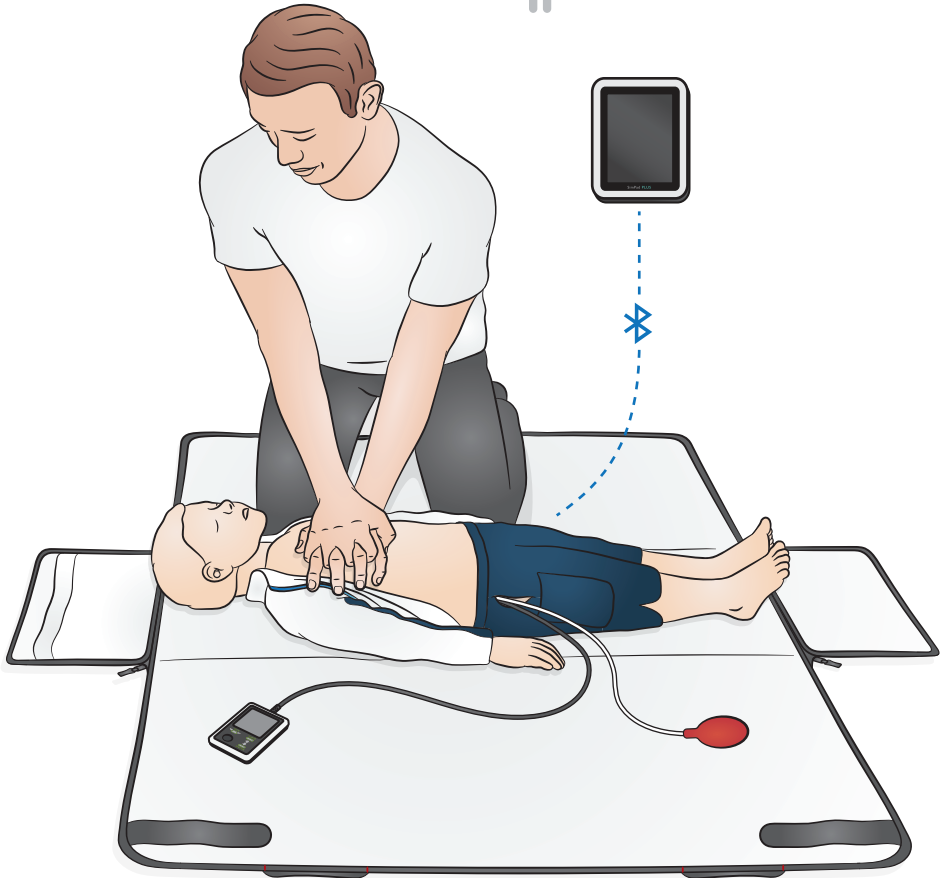


EN

- Connect to SkillGuide with cable.
- Connect to SimPad wirelessly or with a USB-C cable.
- Connect to other devices wirelessly.

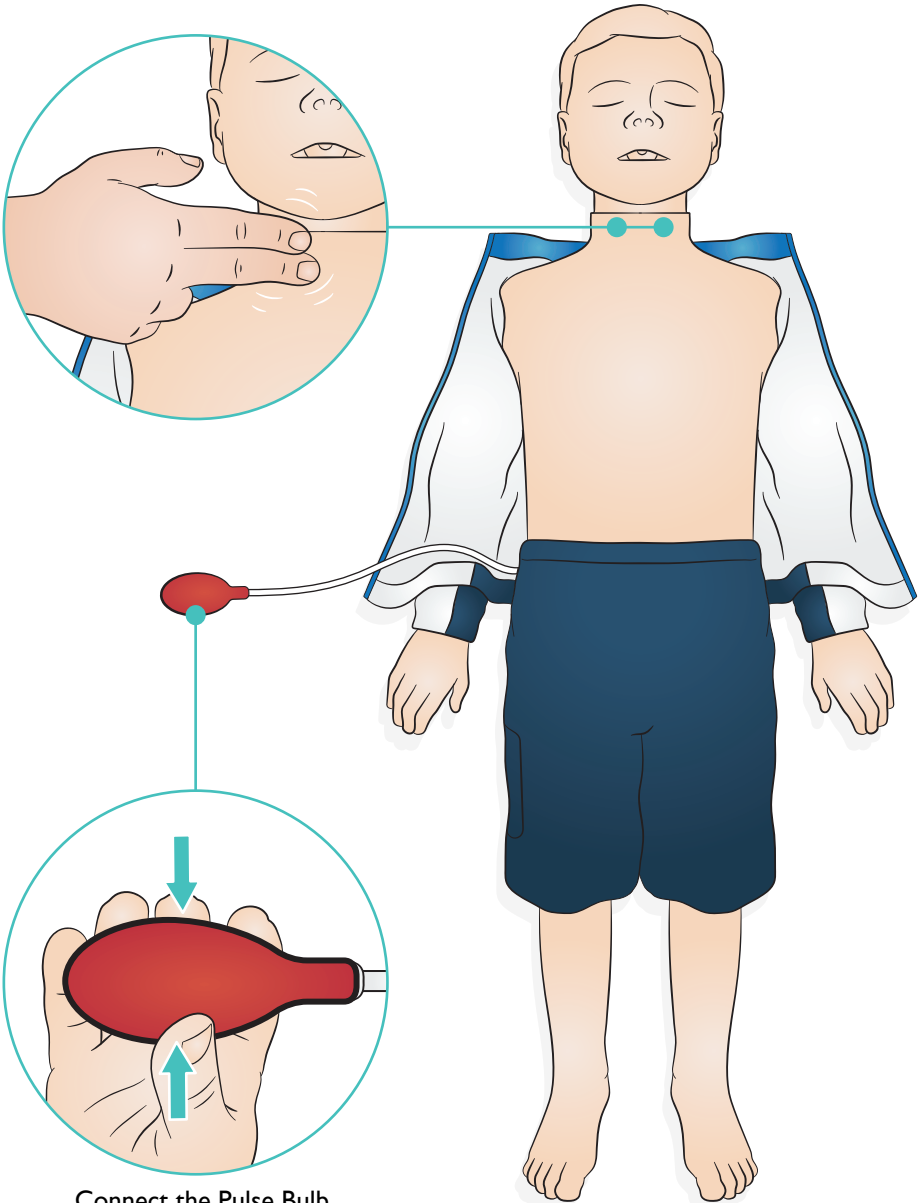




Connect the Pulse Bulb.  
Pump to simulate carotid pulse.


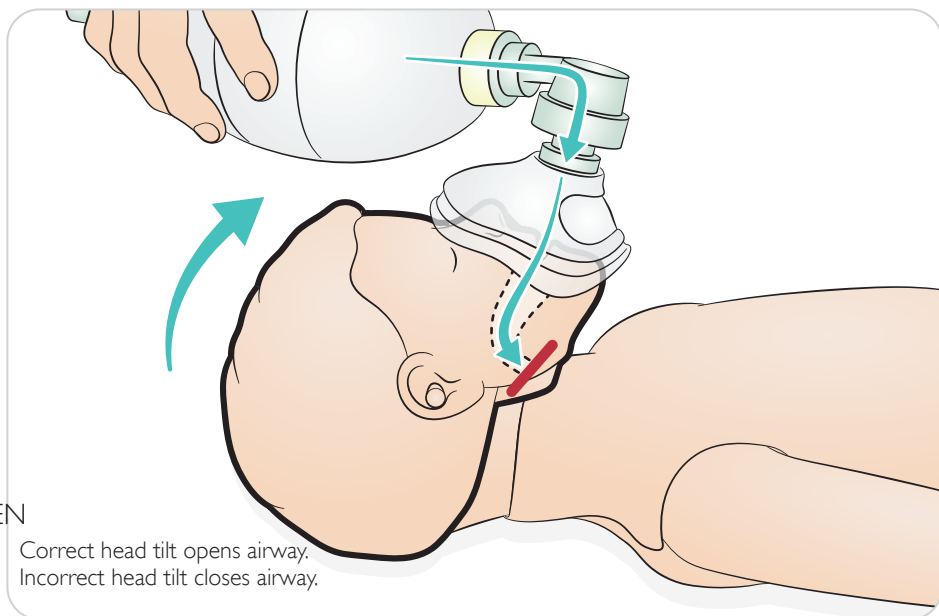
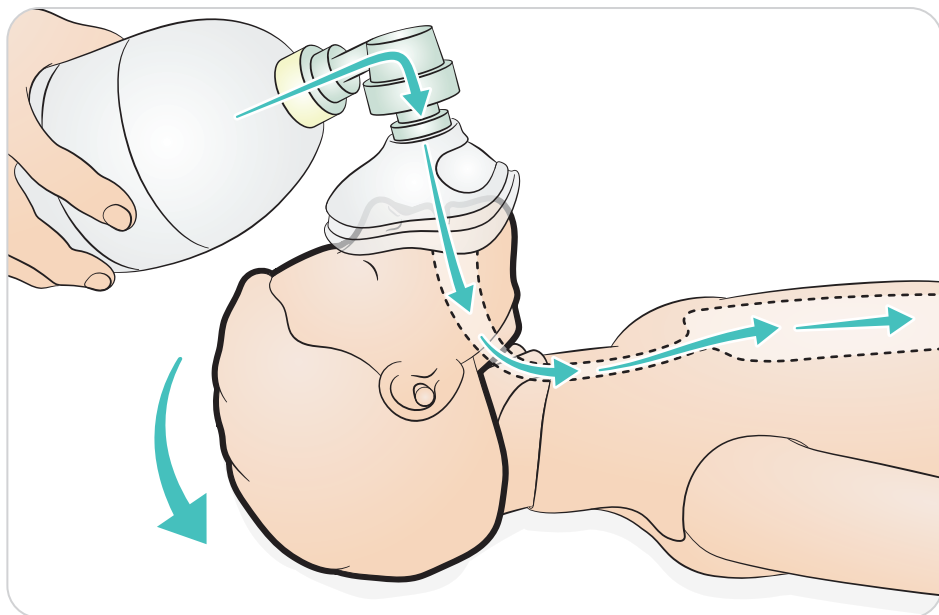


EN

- Correct head tilt opens airway.
- Incorrect head tilt closes airway.




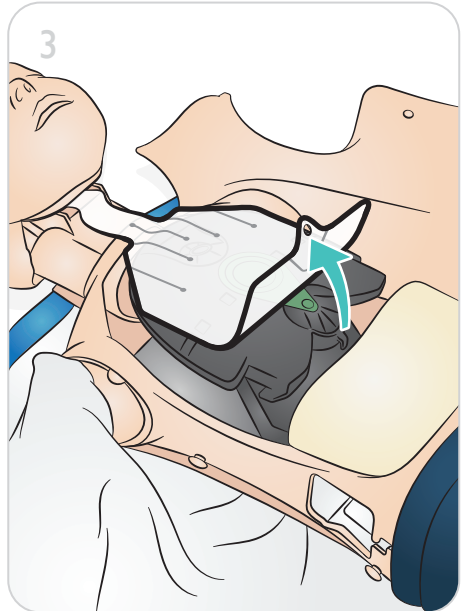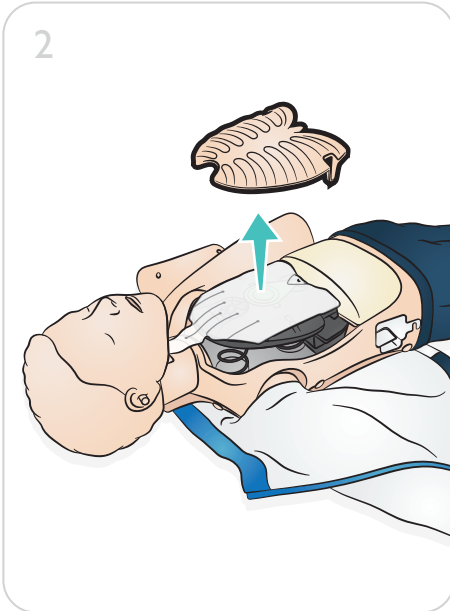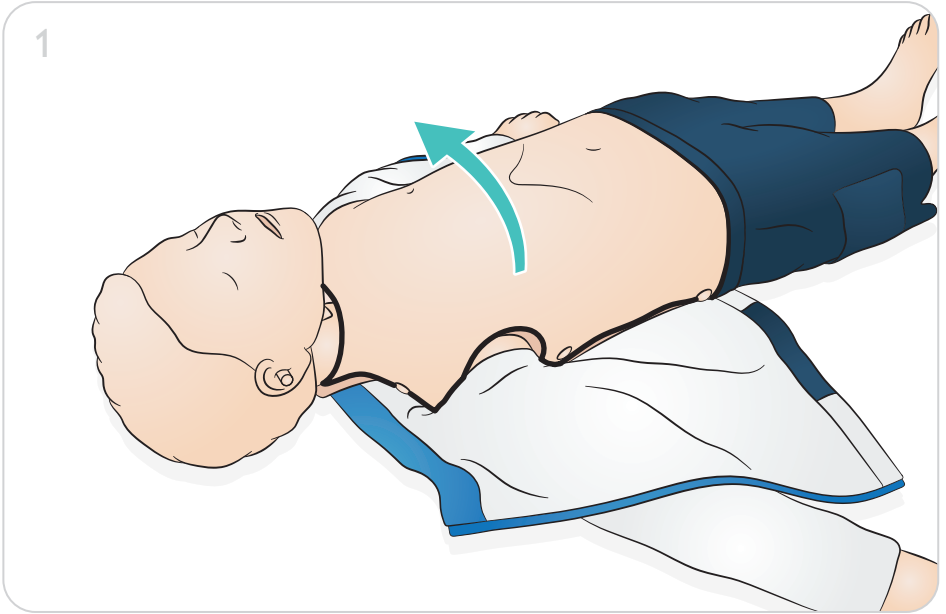

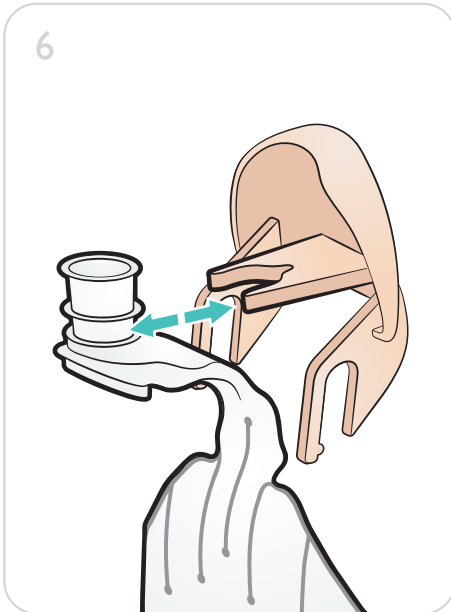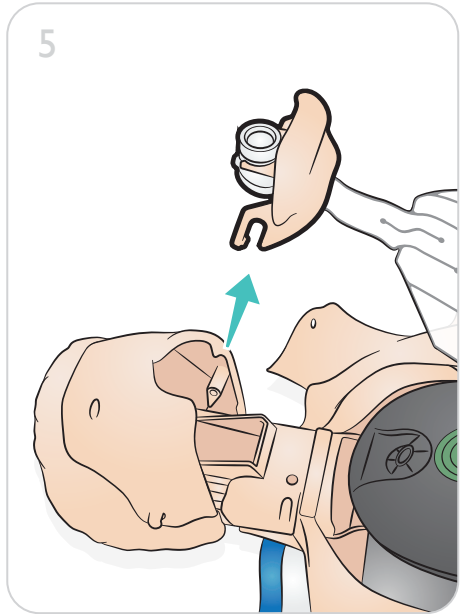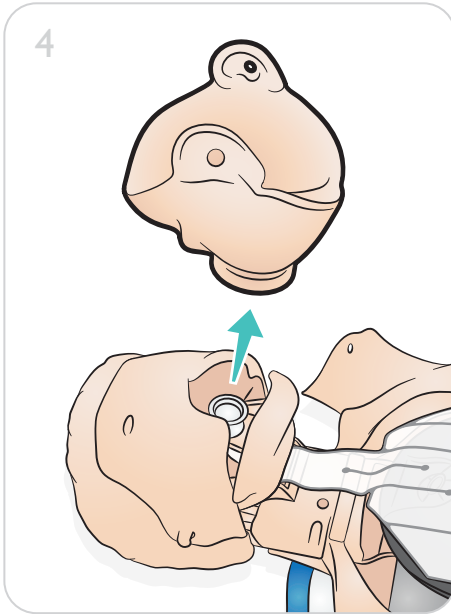





1

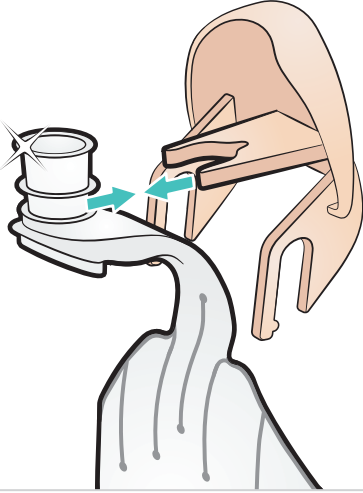


2




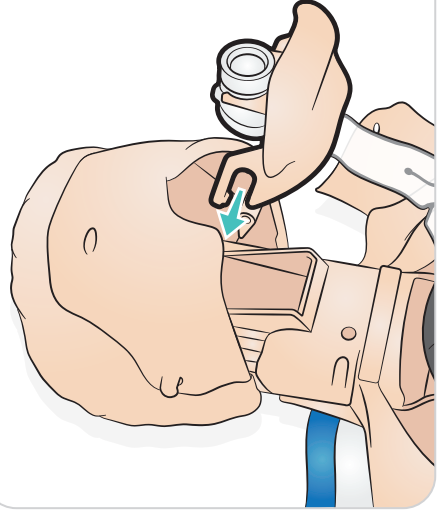



3

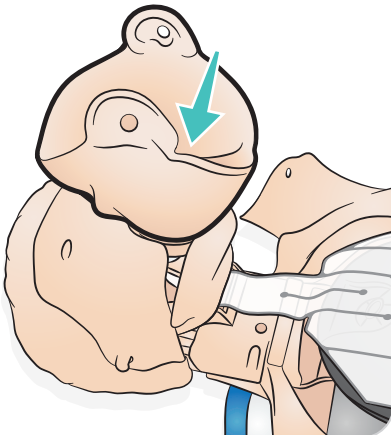

4

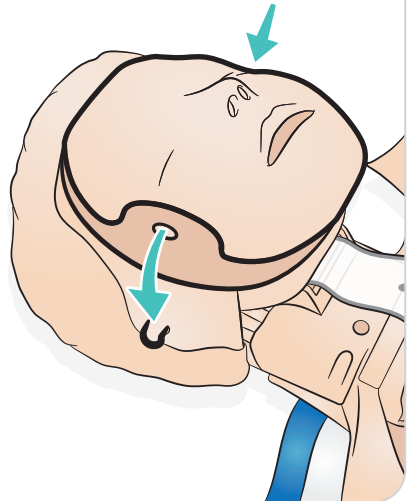

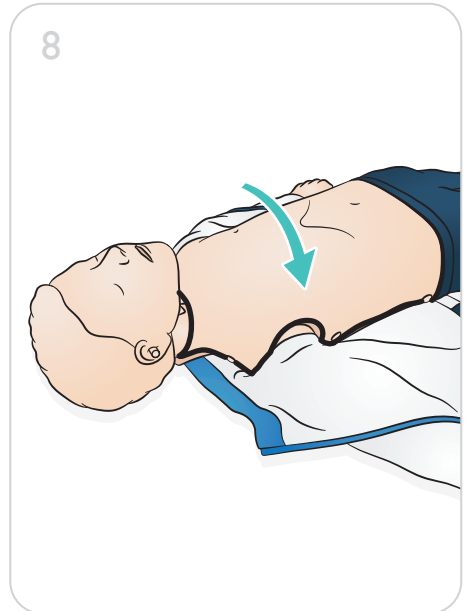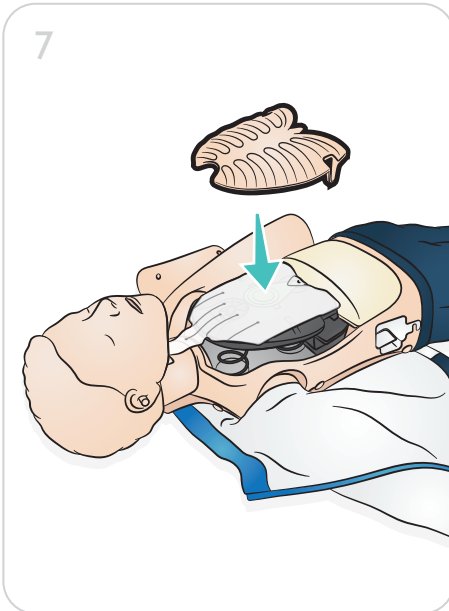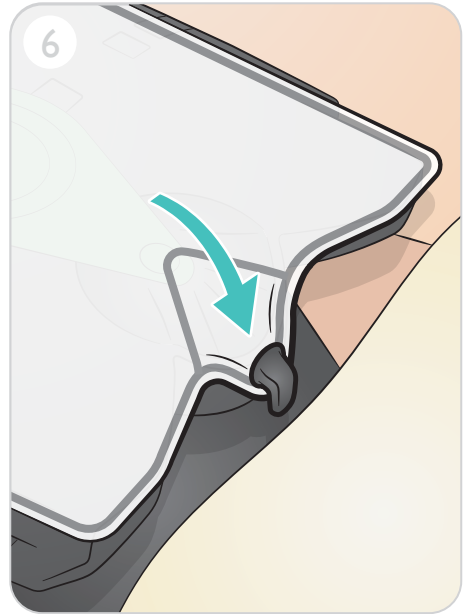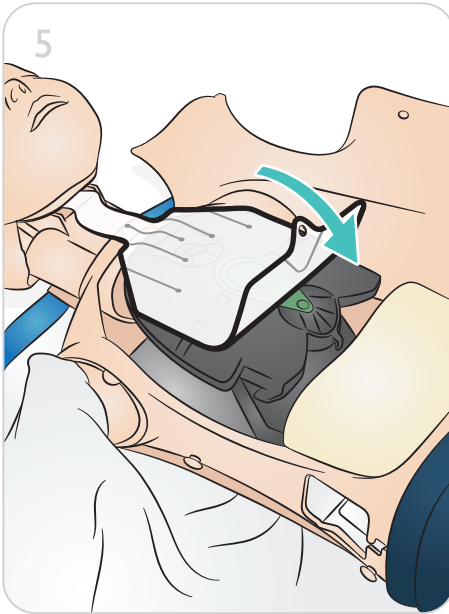

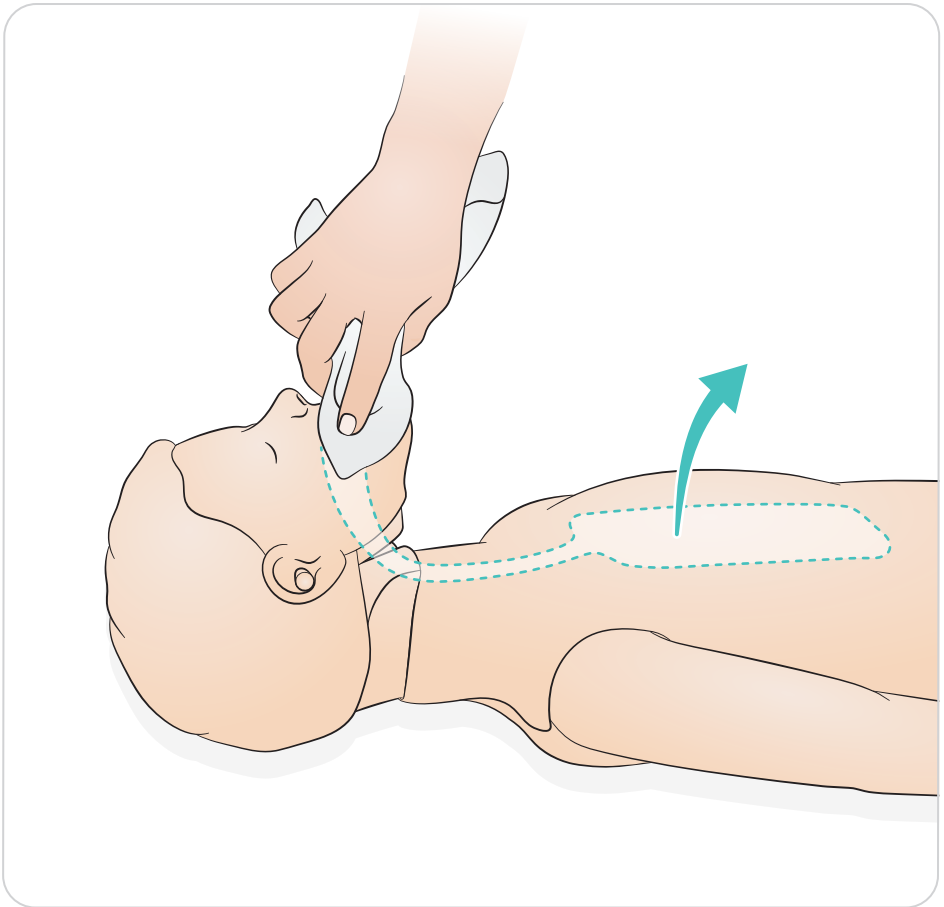

## EN

### Care, Maintenance and Cleaning

- Keep the manikin clean and in a hygienic condition.
- Replace the airway after every class, especially if mouth to mouth resuscitation is practiced.
- We recommend that you use a separate Laerdal Manikin Face for every student.
- If several students use one manikin face, thoroughly sanitize the manikin face after every use.
- Clean all skin parts regularly. Use warm soapy water or Laerdal manikin wipes.
- Use of Manikin Face Shields provides a clean barrier between your lips and the manikin face.

These can discolor the manikin:

- Pigments from lipstick and pens
- Latex gloves
- Using clothes other than what is provided with the manikin originally.

© 2018 Laerdal Medical AS. All rights reserved.

Manufactured in China for:  
Laerdal Medical AS  
P.O. Box 377,  
Tanke Svilandsgate 30, 4002 Stavanger, Norway  
T: (+47) 51 51 17 00

Printed in China

20-12473 Rev C

