

# SkillGuide

## User Guide

English



**Laerdal**

helping save lives

# SkillGuide

---

SkillGuide is a feedback device designed to provide real-time and summative feedback on the quality of CPR performance.



[www.laerdal.com](http://www.laerdal.com)

## Items included

---


SkillGuide and User Guide.



## Connect

---


Connect SkillGuide to the manikin.

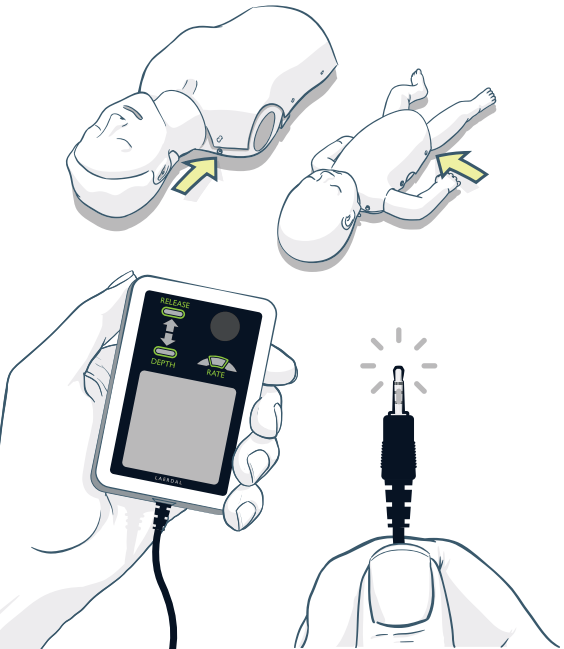

## Switch On

---

Switch on the manikin to activate SkillGuide.

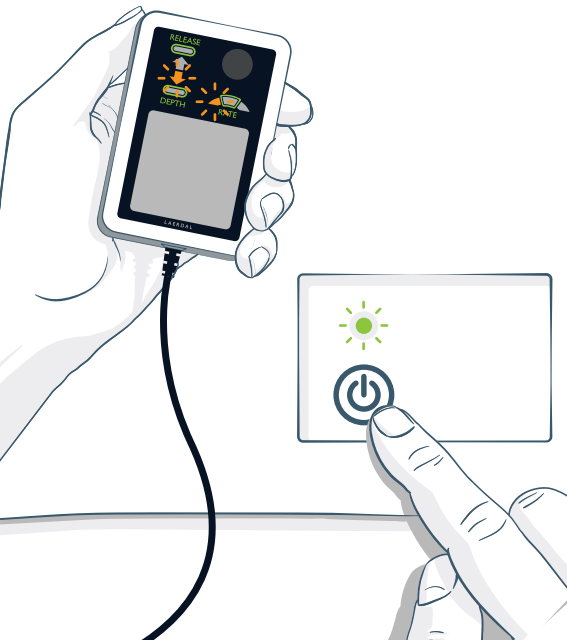

## Start

---


Session timer starts when you begin CPR and stops when you toggle to Debrief Mode. Press and hold the button for one second to reset.



## Toggle Between Modes


---

Press button to toggle between modes

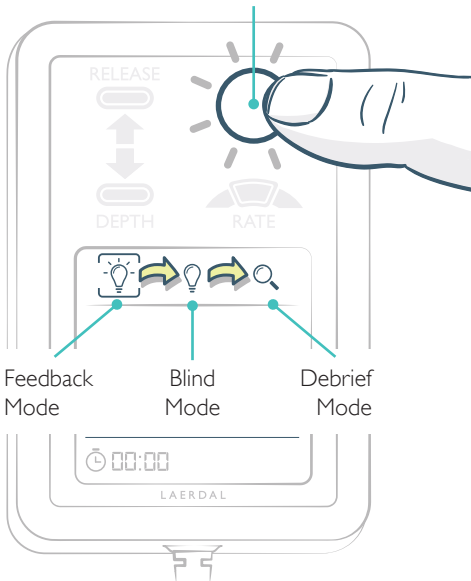

## Feedback Mode

In Feedback Mode the LED indicators show real-time compression feedback. Session time and incorrect hand placement (if present) are displayed on the screen.

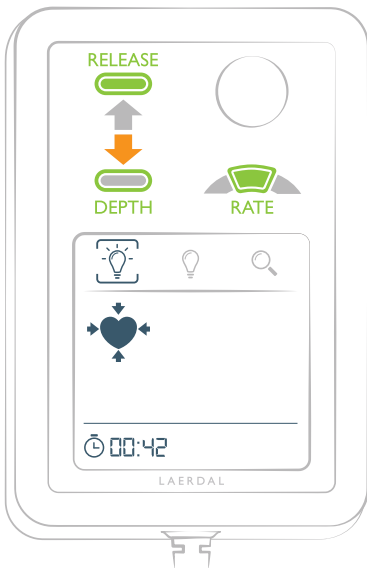





# Feedback Mode

---

## Depth Indicator

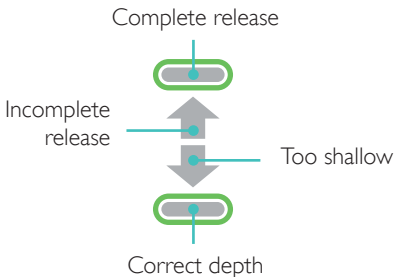


## Compression Rate Meter

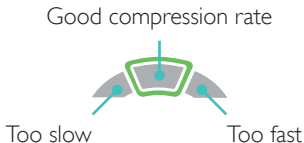

## Blind Mode


---

In Blind Mode no compression feedback is displayed, only the Session Timer.

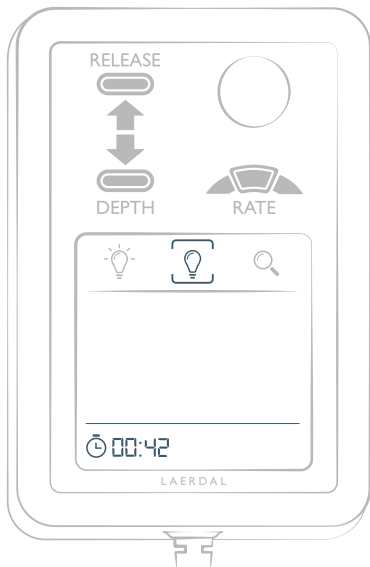

## Debrief Mode

In Debrief Mode the LEDs and the display show a performance summary of the session.

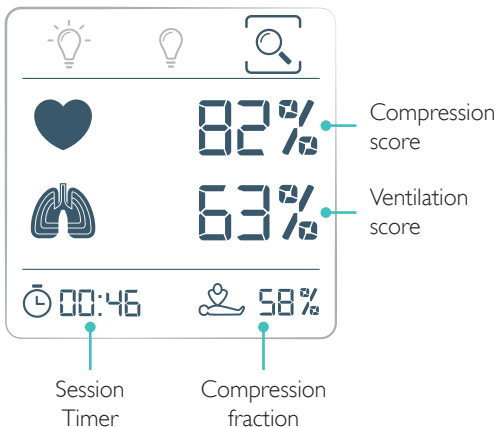

## Debrief Mode

---

### Compression Score

A total score on compressions for the session. Factors used for calculating the score are compression depth, rate, release, and hand position.

In Debrief Mode the indicator lights show the relevant errors for depth, release and rate for the session. The most significant error flashes.

RELEASE



DEPTH



RATE

## Debrief Mode

---

### Hand Position

The heart symbol will indicate if hand position was correct during the session.



Correct  
hand position



Incorrect  
hand position

# Debrief Mode


---

## Ventilation Score

A total score on ventilations. Factors used for calculating the score are ventilation volume and rate.

## Ventilation Volume

The most significant error (if any) is displayed by a short animation.

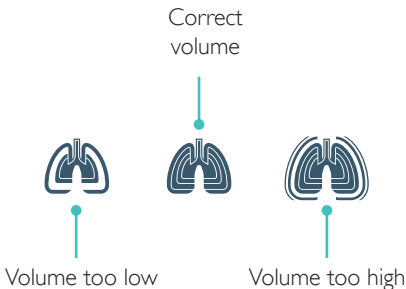

## Debrief Mode

---



### Session Timer

Measures the duration from the first compression or ventilation until the session is stopped. You can stop a session by toggling to Debrief Mode.



### Compression Fraction

Percentage of the session where compressions were given.



For more details on the score calculations, please visit [www.laerdal.com](http://www.laerdal.com)

